

## MIO Infusion Set – Penetration Failures



Incident 1

These failures were caused by improper loading of the Mio trigger mechanism.

See the next page.



Incident 2



Incident 3

When loading the Mio spring mechanism, ensure both left and right triggers are engaged. This will ensure the stylus penetrates at a correct 90 degree angle. Below is an example of where the Mio was not properly loaded. Hold the device at eye level and you can quickly check if it is properly loaded. If it isn't, pull it up until both triggers engage and the white top edge of the device clearly level.



holding tongue not fully engaged



holding tongue fully engaged



resulting asymmetry



Correct this asymmetry by pulling up the lower side until both triggers are fully engaged. Then place on body and simultaneously press the indented release points. This ensures proper stylus angle and entry.